

# HINTERMANN BONE DISTRACTORS



- Useful in foot and ankle surgery
- Flexible positioning
- Accommodates two different K-wire sizes

**MERCIAN**  
Surgical Instruments of Excellence

[www.merciansurgical.com](http://www.merciansurgical.com)

# HINTERMANN DISTRACTORS

The Hintermann Distractor is used in foot and ankle surgery for bone distraction as an alternative to Lamina spreaders. The instrument can be locked and applies distraction across two K-wires placed on either side of a joint.

Two K-wires (either 1.6 or 2.5mm Ø) are introduced into the correct holes in the spreading arms, and penetrate the bone at appropriate position into each of the two bone fragments to be distracted.

The spreader is then distracted, and locked when the required distance between the two bone fragments is reached.

The Hintermann offers flexible positioning with the advantage of not interfering with the procedure.

These distractors are available in two different sizes (15cm or 21cm), and with closed or outspread arms, allowing them to fit different indications.

## Hintermann Distractor with closed arms



Closed arms



1.6mm and  
2.5mm Ø holes

Code	Length	Opening
HA5528-15	15cm/6"	28mm
HA5532-21	21cm/8¼"	32mm

## Hintermann Distractor with open arms - for wider distraction



Open arms



1.6mm and  
2.5mm Ø holes

Code	Length	Opening
HA5028-15	15cm/6"	28mm
HA5032-21	21cm/8¼"	32mm



**MERCIAN**  
Surgical Instruments of Excellence

established over  
45 years

Mercian Surgical Supply Co. Limited

10 Topaz Business Park, Topaz Way, Bromsgrove, Worcestershire B61 0GD. UK.  
Tel: +44 (0) 844 879 1133 Fax: +44 (0) 844 879 1155 Email: info@merciansurgical.com  
www.merciansurgical.com